
# Tulip® GEMS™

Single-Use • CellFriendly®

## Instructions for Use

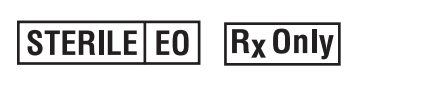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## ENGLISH

**Tulip® GEMS™ Single-Use Instruments**

Single-use Instruments are available in a variety of tip styles, diameters, and lengths, and they are packaged pre-sterilized in clear blister pouches. Each instrument’s diameter is printed on the blister pouch and/or the hub of the SuperLuerLok™. The part number of each instrument is identified on its blister pouch.

The patented SuperLuerLok (SL) is a modification of the “traditional luer lock”. The SL solves the problems associated with structural failure of traditional luer locks by adding strength and rigidity at the connection point. The SL significantly reduces the chance of leakage, creates a static vacuum, and allows a medical instrument to be aggressively manipulated during various medical procedures, most notably during liposuction.

**NOTE:** Before opening a sterile product, visually inspect the seal around the pouch for any openings.

**DO NOT USE** if the blister pouch is not sealed.

Open only in a designated sterile field.

**Indications For Use**

The Tulip Premium Single-Use instrument is intended for use in soft tissue procedures.

**Instructions For Use:**

- Check to ensure the correct style, diameter and length for the desired procedure.
- Make sure the product is opened in a sterile field. Open by pulling and separating each end of pouch.
- Connect the instrument by twisting the hub onto a sterile luer lock syringe.
- For suction procedures, prime the instrument with saline to eliminate air space and increase suction efficacy.
- Note: There is a flat surface on the SuperLuerLok of each Tulip instrument that displays the Tulip logo. This flat surface is positioned opposite the opening in the tip of the instrument, identifying which direction the tip opening is facing during surgery.

**Tips For Handling The Disposable Instrument**

Be careful to avoid splashing extracted tissue or fluid onto operating room personnel.

Remnants of liquids and oils on the mating surfaces of syringes and cannulas can interfere with a good seal.

To clear a plugged instrument, insert the instrument opening into a splash-proof container and clear the contents from the instrument with quick, short, in and out strokes. Flush with saline.

**IF SUCTION DECREASES, CHECK TO ENSURE THERE IS NO LOSS OF VACUUM AT THE INSTRUMENT-TO-SYRINGE CONNECTION POINT.**

### ! WARNINGS

**SINGLE-USE INSTRUMENTS ARE NOT TO BE REPROCESSED OR REUSED.**

This device will not, in and of itself, produce significant weight reduction.

This device should be used with extreme caution in patients with chronic medical conditions such as heart, lung, or circulatory disease, diabetes, or obesity.

The volume of blood loss and endogenous body fluid loss may adversely affect intra and/or postoperative hemodynamic stability and patient safety.

The capability of providing adequate, timely fluid replacement is essential for patient safety.

When finished using the instrument, dispose of properly in accordance with biohazard regulations.

**PRECAUTIONS**

This device is designed to reform soft tissue through small incisions.

Use of this device is limited to those physicians who have attained proficiency in suction lipoplasty.

Results of this procedure will vary depending upon patient age, surgical site, and experience of the physician.

Results of this procedure may or may not be permanent.

The amount of fat removed or injected should be limited to that necessary to achieve a desired effect.

**Tulip GEMS Single-Use Accessories Indications For Use**

The Tulip Premium Single-Use Accessories are intended for use in soft tissue procedures.

**NOTE:** Before opening a sterile accessory, visually inspect the seal around the pouch for any openings.

**DO NOT USE** if the blister pouch is not sealed.

Open only in a designated sterile field.

**Tulip GEMS Single-Use Anaerobic Transfers™**

Tulip Single-Use Anaerobic Transfers are made from clear ABS copolymer plastic. The unique “wing” design of the transfer creates a grip that allows for quick and easy attachment to all sizes of standard luer syringes.

**Indication For Use**

Indicated for the transfer of tissue and/or fluid from one syringe to another.

**Instructions For Use:**

- Remove the Anaerobic Transfers from their sterile packaging.
- Attach syringes by rotating them onto the luer lock openings of the transfer.
- Test for a secure connection, making sure the Transfer is seated tightly onto the luer locks of the syringes.
- Transfer tissue and/or fluid from syringe to syringe using a pulling technique on the receiving syringe.

**Tulip GEMS Single-Use Johnnie Snap™ Syringe Lock**

The Johnnie Snap syringe lock is designed to maintain vacuum at variable pressure levels, from low to high. The Johnnie Snap comes in two sizes, 20cc and 60cc, and fits the following syringes:

- 20cc BD Syringes
- 20cc Monoject Syringes
- 60cc BD Syringes
- 60cc Monoject Syringes

**Indication For Use**

Indicated for use as a syringe lock.

**Instructions For Use:**

- Remove Johnnie Snap and two o-rings from its sterile packaging. The o-ring is only included and to be used with the 20cc Johnnie Snap.
- Remove appropriate syringe from sterile packaging.
- Pull the syringe plunger out to its full extent.
- Install the Johnnie Snap into the groove created by the plunger flanges. Make sure that the markings on the Johnnie Snap face the opposite direction from the markings on the syringe. (For the 20cc Monoject syringe, twist off the top tab of the Johnnie Snap prior to installation.) Place one o-ring around the top of the plunger to help hold the Johnnie Snap in place. The second o-ring is a spare.
- Push the plunger (now loaded with the Johnnie Snap) back into the syringe, using your finger to push down each tooth as needed to clear the collar.
- Once the plunger is seated completely back into the barrel, align the Johnnie Snap under the measuring grid on the syringe barrel and attach desired suction instrument. Prime instrument with saline to eliminate the air space.
- Insert the instrument into the patient. (Use small puncture openings rather than incisions for optimal closed system vacuum conditions.)
- Grip the plunger, placing your thumb over the top edge of the Johnnie Snap, and pull the plunger back, allowing the teeth to snap out and anchor the plunger in the opposite direction.

**Tulip GEMS Single-Use Johnnie Lok™ Syringe Lock**

**Indication For Use**

Indicated for use as a syringe lock on a 12cc Monoject syringe.

**Instructions For Use:**

- Remove Johnnie Lok from its sterile packaging.
- Remove appropriate syringe from sterile packaging.
- Attach the Johnnie Lok to the collar of the syringe barrel, rotating the fins of the syringe plunger so that the Johnnie Lok slips easily into it place, with the marking facing up.
- Attach the desired instrument firmly to the syringe.
- Prime the instrument with saline to eliminate air space.
- Insert the instrument into the patient. (Use small puncture openings rather than incisions for optimal closed system vacuum conditions.)
- Pull plunger back to desired position and twist the plunger in either direction until it clicks and locks.
- Release the plunger by twisting it in the opposite direction.

### ! WARNINGS

**Dispose of single-use instruments in accordance with biohazard regulations.**

**PRECAUTIONS**

Use of these devices is limited to those physicians who have attained proficiency in soft tissue procedures.

- PREMIUM SINGLE USE ITEMS ARE NOT DESIGNED TO BE REPROCESSED OR REUSED. DO NOT REUSE.**

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