
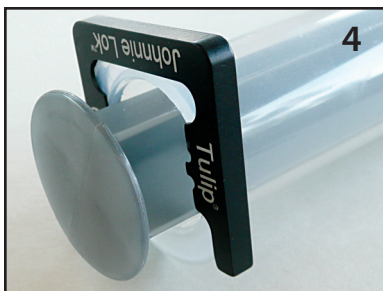


## Tulip Johnnie Lok™ Instructions

The Johnnie Lok slips onto the collar of the syringe barrel. The plunger is drawn back to the desired cc for precision measurement on the syringe. Then, with a small twist, teeth on the lock bite into the plunger flanges and lock the plunger securely to the barrel holding the liposuction syringe under vacuum for precision fat extraction.

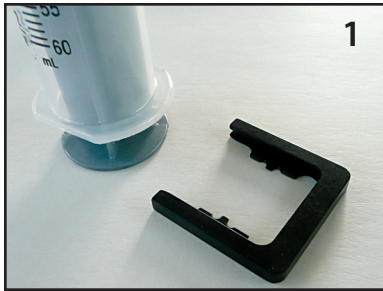

1. The Johnnie Lok fits onto the collar of the syringe barrel with the lettered side facing the bottom end of the syringe barrel.
2. Slip the Johnnie Lok onto the collar of the syringe barrel.
3. Push the Johnnie Lok all the way onto the collar until it seats.
4. Make sure the plunger is aligned properly so the teeth of the lock does not contact the flanges of the plunger when the Lok is slipped over the collar.
5. Pull back the plunger on the syringe to the desired cc of extraction.
6. Twist the plunger to lock the plunger flange into the toothed notch. The notch bites into the flange rendering the plunger immobile.
7. You may twist the plunger in either direction as there are notches for each side. Untwist to release. This action may be repeated with the syringe.